Manufacturer's Recommendations for Alternate Dental CBCT QA Program Genoray: Model Papaya 3D- Papaya Plus 3D Table 3A Computed Tomography QC Requirements

| | | T | 1 | T |
|------|--|--------------------|--|---|
| Item | Required Test or Procedure | Frequency | Substitute Test or Procedure | Standard |
| 1 | Equipment Function "Indicators, Mechanical & other Safety Checks | Daily | User Manual Chapter 1 Introduction of Product Section 1.2-1.6 Pages 6-11 | Must work properly. |
| 2 | Film Processing QC Testing | Daily | (Appendix B) None, not applicable | None, not applicable |
| 3 | CT Number for Water | Daily | Quality Control Test Procedure Section 3.3 Noise & CT Number of Acrylic Pages 8-9 (Appendix A) | Average Value 40 ~ 200 Std Value 40 ~ 100 |
| 4 | Field Uniformity | Daily | Quality Control Test Procedure Section 3.3 Noise & CT Number of Acrylic Pages 8-9 (Appendix A) | Average Value 40 ~ 200 Std Value 40 ~ 100 |
| 5 | Laser Film Printer QC | Weekly | None, not applicable | None, not applicable |
| 6 | Low Contrast Resolution | Initial & Annually | Quality Control Test Procedure Section 3.5 Spatial Resolution for High and Low Contrast Object Pages 12-14 (Appendix A) | Low Contrast Resolution 6 mm |
| 7 | High Contrast Resolution | Initial & Annually | Quality Control Test Procedure Section 3.5 Spatial Resolution for High and Low Contrast Object Pages 12-14 (Appendix A) | Spatial Resolution 0.6 mm |
| 8 | Noise | Initial & Annually | Quality Control Test Procedure Section 3.3 Noise & CT Number of Acrylic Pages 8-9 (Appendix A) | Average Value 40 ~ 200 Std Value 40 ~ 100 |
| 9 | Scan Localization Light Accuracy | Initial & Annually | Technical Manual-Appendix Section 4- Alignment and Calibration Panoramic Laser Alignment Section 4.6 Step 6 Pages 22 (Appendix C) | Install 'Laser Alignment jig' on the Ball phantom and adjust laser position, refer to Figure 4.10. Align the three lasers to laser alignment jig, according to the drawings. Must align with laser alignment jig. |

| 10 | Medical Physicist's QC Survey | Initial & Annually | Same as alternate Dental CBCT | NJAC 7:28-22.10 |
|----|--|--------------------|-------------------------------|--------------------|
| 11 | Medical Physicist's Quality Assurance Program Review | Initial & Annually | Same as alternate Dental CBC | NJAC 7:28-22.4(a)7 |