Manufacturer's Recommendations for Alternate Dental CBCT QA Program Air Techniques Model: ProVecta 3D Prime & ProVecta 3D Prime Ceph Table 3A

Computed Tomography QC Requirements

	Required Test or			
Item	Procedure	Frequency	Substitute Test or Procedure	Standard
1	Equipment Function:	Daily	ProVecta 3D Prime and	Must work properly
	Indicators, Mechanical		ProVecta 3D Prime Ceph	
	& other Safety Checks		Operating Instructions	
			Chapter 2 (Safety): Pages: 5 – 8	
			Chapter 10 (Maintenance):	
			Pages: 54 – 55	
			(Appendix A)	
2	Film Processing QC Testing	Daily	None – Not Applicable	None – Not Applicable
3	CT Number for Water	Daily	Quality Control (QC) manual	Test with Hounsfield
			Provecta 3D Prime	phantom
				HU Range: - 200 to +200
			Chapter 2.1 Pages: 3 – 6	
	D' 1111 'C '.	D 11	(Appendix B)	TOTAL .
4	Field Uniformity	Daily	Quality Control (QC) manual Provecta 3D Prime	The maximum mean value must not differ more than
			Provecta 3D Prime	400 HU from the minimum
			Chapter 2.2 Page: 6	mean value.
			(Appendix B)	mean varae.
5	Laser Film Printer QC	Weekly	None – Not Applicable	None – Not Applicable
6	Low Contrast	Initial & Annually	Quality Control (QC) manual	At least 8mm PE cylinder
	Resolution		Provecta 3D Prime	is visible
			Chapter 2.3 Pages: 7 – 8	
	TT' 1 G	T 11 1 0 1 11	(Appendix B)	
7	High Contrast Resolution	Initial & Annually	Quality Control (QC) manual Provecta 3D Prime	At least 1 mm drilled air
	Resolution		Provecta 3D Prime	hole visible
			Chapter 2.4 Pages: 7 – 8	
			(Appendix B)	
8	Noise	Initial & Annually	Quality Control (QC) manual	Consistency test DIN
			Provecta 3D Prime	6868-15
				Initial Contrast -Noise
			Chapter 2.6 Page: 12 – 14	value +/- 40%
			(Appendix B)	
				The test is passed when all
				parameters are displayed in
9	Scan Localization	Initial & Annually	Quality Control (QC) manual	green. Evaluated by ServiceTool
9	Light Accuracy	Illitial & Allitually	Provecta 3D Prime	software automatically.
	Ligit Hoodiacy		110,000 3D Time	software automatically.
			Chapter 2.7 Page: 15 – 17	This point is passed when
			(Appendix B)	all relevant parameters are
				outlined green.
10	Medical Physicist's	Initial & Annually	See N.J.A.C. 7:28-22.10(a)	N.J.A.C. 7:28-22.10(a)
	QC Survey			
11	Medical Physicist's	Initial & Annually	See N.J.A.C. 7:28-22.10(a)	N.J.A.C. 7:28-22.4(a)7
	Quality Assurance			
	Program Review			